

AOSE ImPACT TEST INSTRUCTIONS

1. Set aside 30 minutes in a quiet room with no distractions to take the test
2. No headphones or cell phone use during the test
3. Turn off any televisions, radio, or anything else that can produce background noise.
- 4. AVOID ALL INTERRUPTIONS**
5. **You must use a standard external mouse.** You may not use a finger mouse pad (ie: laptop mouse), a Track Mouse, or anything other than a standard mouse
6. Read the directions carefully
7. Do not allow others to take your test – this baseline exam is intended to measure your brain’s normal ability.
8. Do not write answers down – this will provide inaccurate baseline scores
9. Google Chrome is not a supported browser for the ImPACT test, please use Internet Explorer 6.0 and above, or Firefox 1.5 or above, Safari for the MAC running OSX 10.2 and above
10. You must have Macromedia Flash Player 10.0 or newer. You can download Flash Player at www.adobe.com
11. If you are running Windows 7, make sure power management is set to High Performance – otherwise performance will be slowed, negating test scoring.
12. To take the baseline test, go to <https://www.impacttestonline.com/testing/>

Enter AOSE’s Customer code: **VC7QQXDJVD** and click “Launch Baseline Test.” Be sure your pop-up blocker is turned **off** as the Baseline Exam will appear in a pop-up window. Results will automatically be generated to the AOSE database.